



! use 00905 cable ties at all lines
 tüm linelerde 00905 kablo bagi kullaniniz

| | | |
|---|--|----------------------|
| F1000000 F1000000 Pediatric pack 1 | All details Article-No 70105 | serial 106 |
| | FB-1224 / Valid fr. 2 | page 1 / 6 |

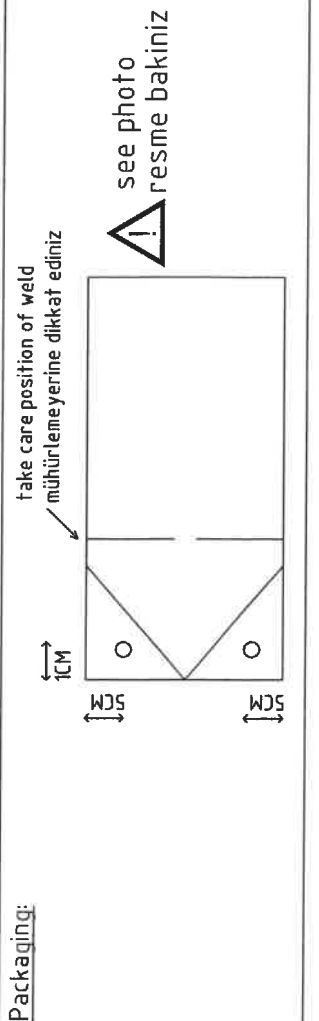

Açılışı sağa doğru / clamps open right:

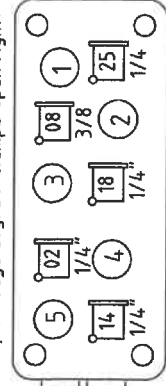


- 1 Plate 1031241
- 1 Clamp 1031239
- 4 Clamps 1031240
- 4 Rivets 1035753

double tape
çift bant



- 1 Sterile Bag
- 0,25m Band
- 1 BE-Label small

- 09041
- 07515
- 04310

Tyvekband

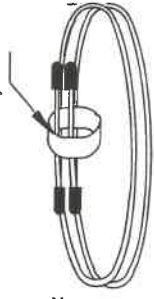

close sterile bag with gas tape and additional 07515 velcroband. Must be tight sterile torbayı gasband ve 07515 ile kapatınız 07515 velcroband çok sıkı olmalıdır.

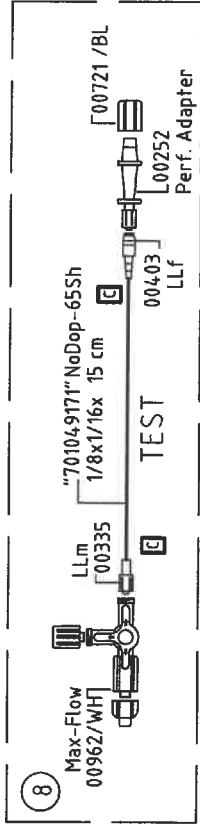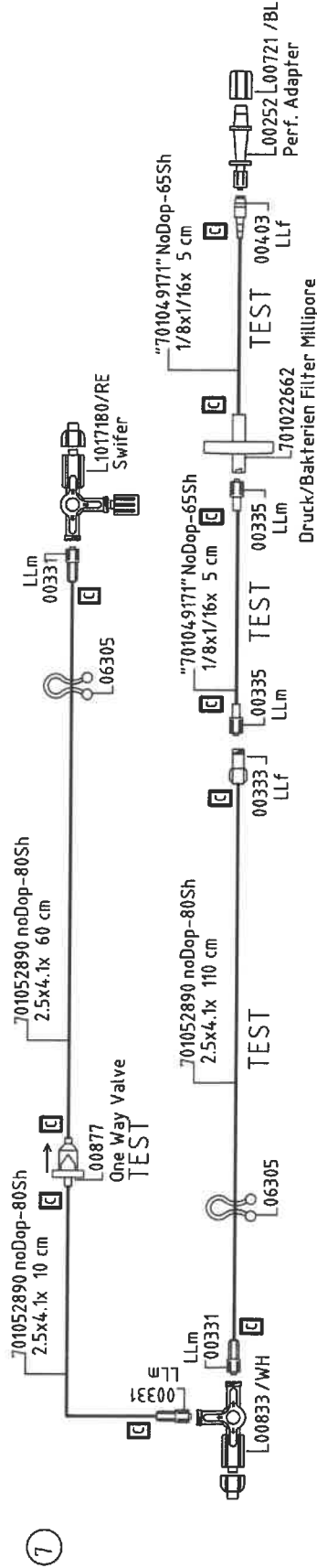


- 1 Sterile Bag 04192
- 1 Label small 04137

Packaging:


Pediatric Pack 1

All design, fr
Article-No.
701053
FB-1224 / Vet
Valid fr. 2018

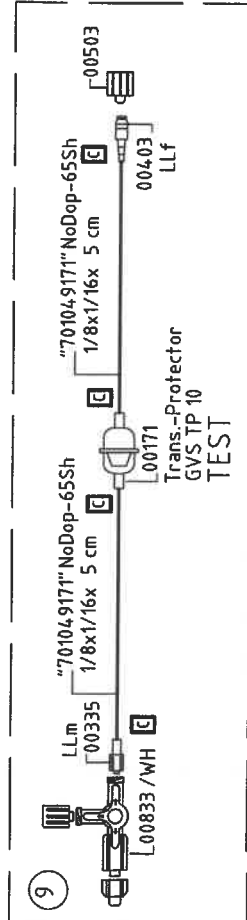


1 Sterile Bag
1 Label small

04136
04137 Exp. date

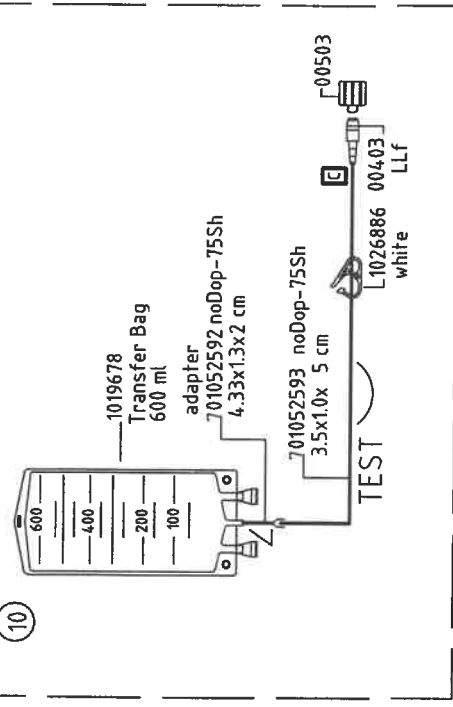

1 Sterile Bag
1 Label small

04136
04137 Exp. date

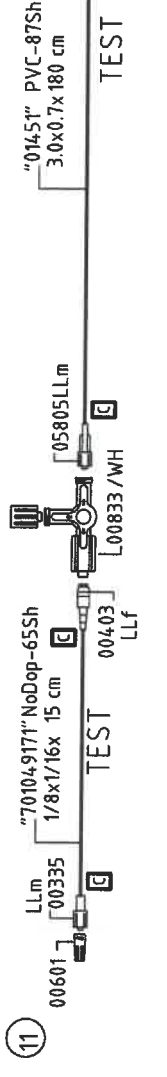

1 Sterile Bag
1 Label small

04150
04137 Exp. date

Packaging:


Counters: 0
Filter: 0

All items reserved
Article-Nr 70105
FB-1224 / Valid fr. 2


Article-Nr 70105
Page 3 / 6

Pediatric pack 1

12
00905

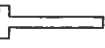


13

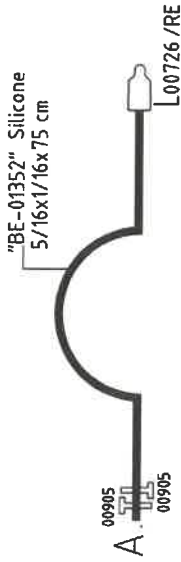

15




14

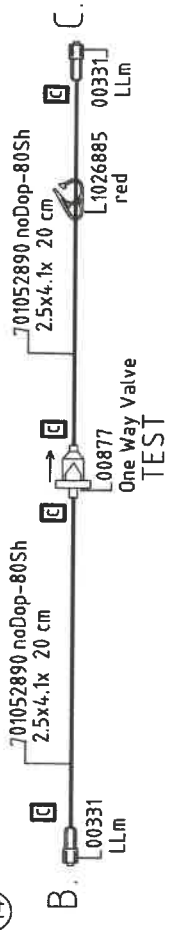


! Fix on reservoir with two gastape - see photo
iki gasbant ile rezervuara sabitleyiniz - resme bakiniz

16

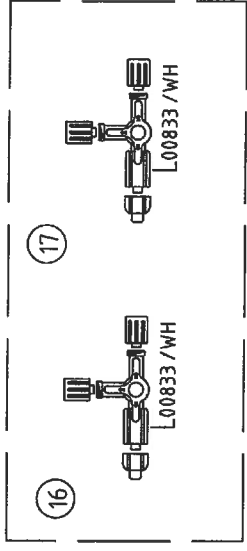

1 Sterile Bag
1 Label small
04.191
04.137
Exp. date

18

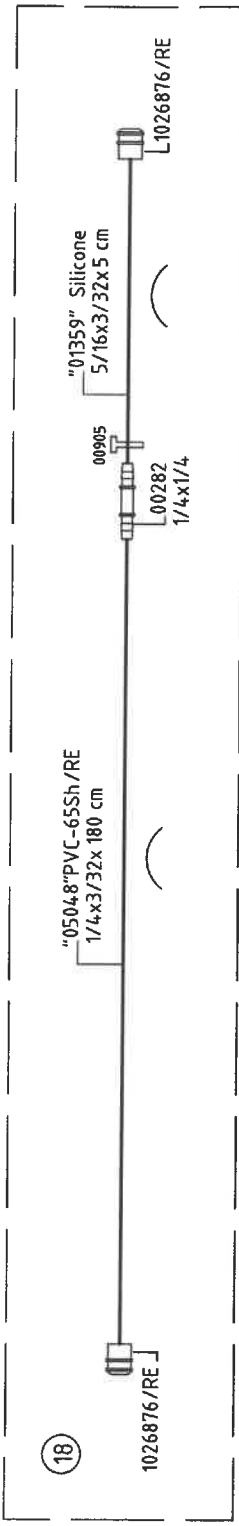

1 Sterile Bag
1 Label big
04.150
04.148
Moisture Trap

Packaging:

Faustro fitv

All design reg


Article-No
7010539

FB-1224 / Ver
Valid fr. 2018-


Pediatric Pack 1

16


/ 6

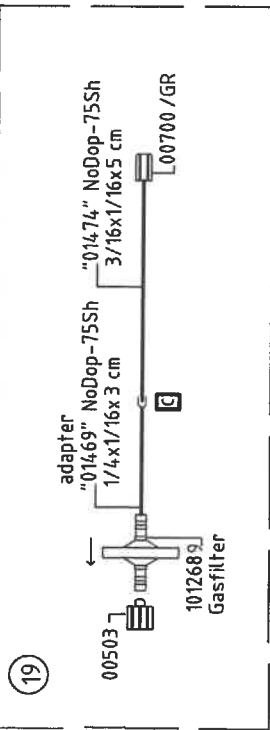


1 Sterile Bag 04136
1 Label small 04137 Exp. date

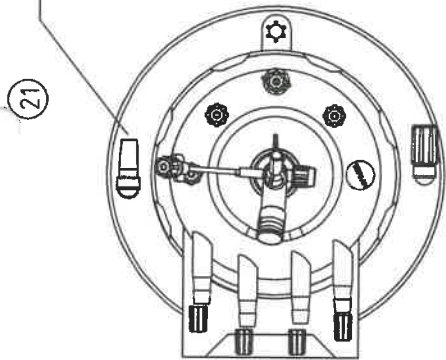


21
"70104,1560"
Blind Tube 1/4x1/16x4cm

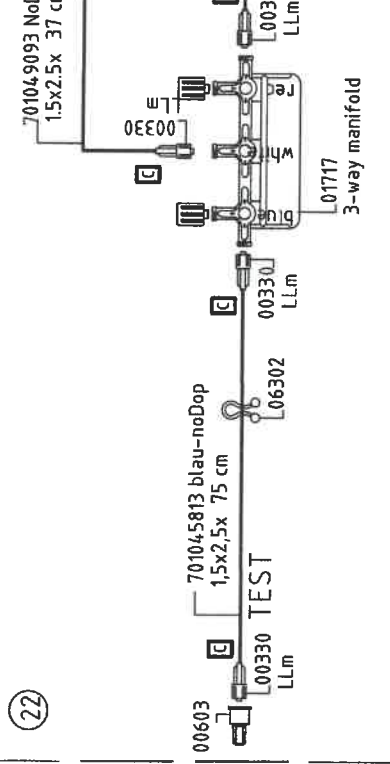

3 Lot Label, grey 1020612 enclosed not glued
yapistiramayiniz

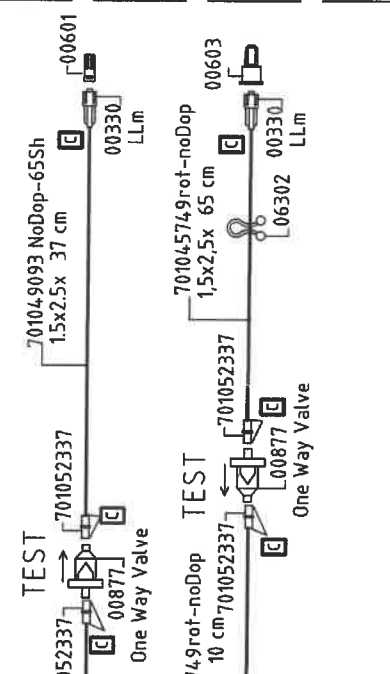

1 Sterile Bag 04136
1 Label small 04137 Exp. date

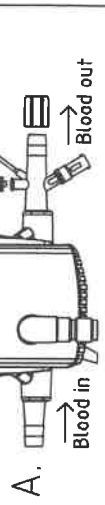

| Packaging: | | Country | Article-N | Page |
|------------|-----------------------------------|---------|-----------|-------|
| 1 | Tray (80x40x21,5cm) | | 70104 | 5 / 6 |
| 1 | Insert HC-Res. | | | |
| 1 | Inlay Quadrox-i | | | |
| 2 | Velstrap short | | | |
| 1 | Velstrap long | | | |
| 1 | Tyvekcover | | | |
| 3 | Label big | | | |
| 1 | Label big | | | |
| 1 | Label big JCTS | | | |
| 1 | BE-Label middle | | | |
| 1 | BE-Label big | | | |
| 1 | Paper between | | | |
| 1 | Carton | | | |
| 1 | IFU BMU Sensor and Cell 104.104.9 | | | |
| 1 | IFU Transfer Bag 701055018 | | | |
| 1 | IFU Quadrox 30000 701045686 | | | |
| 1 | IFU VHK 11000/31000 701066415 | | | |
| 1 | IFU Bioline 34.0035 | | | |
| 1 | IFU HLM Pack 04.248 | | | |
| 1 | Tray (80x40x21,5cm) | | | |
| 1 | Insert HC-Res. | | | |
| 1 | Inlay Quadrox-i | | | |
| 2 | Velstrap short | | | |
| 1 | Velstrap long | | | |
| 1 | Tyvekcover | | | |
| 3 | Label big | | | |
| 1 | Label big | | | |
| 1 | Label big JCTS | | | |
| 1 | BE-Label middle | | | |
| 1 | BE-Label big | | | |
| 1 | Paper between | | | |
| 1 | Carton | | | |
| 1 | IFU BMU Sensor and Cell 104.104.9 | | | |
| 1 | IFU Transfer Bag 701055018 | | | |
| 1 | IFU Quadrox 30000 701045686 | | | |
| 1 | IFU VHK 11000/31000 701066415 | | | |
| 1 | IFU Bioline 34.0035 | | | |
| 1 | IFU HLM Pack 04.248 | | | |

Pediatric pack 1